

# SATISFYER**MEN**

## Instructions for use

### Satisfyer Men – Neutral lubricant

#### Satisfyer Men lubricants

Satisfyer Men lubricants are medicinal products for use on the skin. They have an oil and fat-free formula, and score points thanks to their select raw ingredients, monitored manufacturing processes and comprehensive testing. Satisfyer lubricants are tested for compatibility with the skin and mucous membranes, and are verifiably condom-safe.

All lubricants in the Satisfyer Men range meet the strict requirements of the European Directive 93/42 EEC on Medical Devices and the German Medical Devices Act.

Satisfyer Men lubricants are water-based. This means that they are easily rinsed off with water, leaving behind no residue or sticky feeling on the skin. They are ideal for use with all Satisfyer Men products, but can also be used with other toys or during intercourse.

#### How Satisfyer lubricants work

Satisfyer lubricants make using your Satisfyer Men simple and friction-free. They have a lubricating effect which enables you to quickly penetrate the soft shell of your toy and immediately start to enjoy using it. The lubricant ensures a realistic feel and intensive lubrication. In addition, the colourless and tasteless formula provide a natural enjoyment experience.



#### Applying Satisfyer Men lubricants

The practical bottle enables you to control the application. Place a peanut-sized amount of lubricant on your penis and a small amount on your toy. Spread the lubricant evenly over your erect penis and slide it gently into the masturbator. For intercourse, drip lubricant onto the penis, the condom rolled over the erect penis, or your partner's body part and spread using your fingers. For hygiene reasons, it is recommended that you wash your hands thoroughly before and after use. When using with condoms, it is also imperative that you follow the instructions for use that are specified on the condom packaging.



#### Removing Satisfyer Men lubricants

Satisfyer Men lubricants can simply be washed off with water, leaving no residue, whether from your skin, mucous membranes or toys. For hygiene reasons, it is recommended that you clean your toy thoroughly before and after use. The Satisfyer Men Disinfectant Spray is perfect for this task.

#### Further advice for use during intercourse

##### Contraception and protection against infection

Satisfyer Men lubricants are not a form of contraception. They do not contain spermicides and do not offer any protection against infection.

#### When trying to conceive

Gynaecologists recommend that you should use lubricants sparingly or abstain from using them completely when trying to conceive as they may hamper conception.

#### Further advice on intolerances

Before using this product, if you need to determine whether the products contain any ingredients that may not be tolerated or that could cause an allergic reaction, you should refer to the information detailed on the product label that is listed under 'ingredients (INCI)'. If using the product results in skin irritation, please remove the product following the instructions on the packaging and consult a doctor if necessary.



#### Contents

Water, glycerin, hydroxyethyl cellulose, phenoxy ethanol, allantoin, benzoic acid

#### Manufacturer



Amor-Gummiwaren GmbH  
August-Rost-Str. 4  
99310 Arndtstadt, Germany



CE 0197